


# Prolapse Harness Instructions

#560000

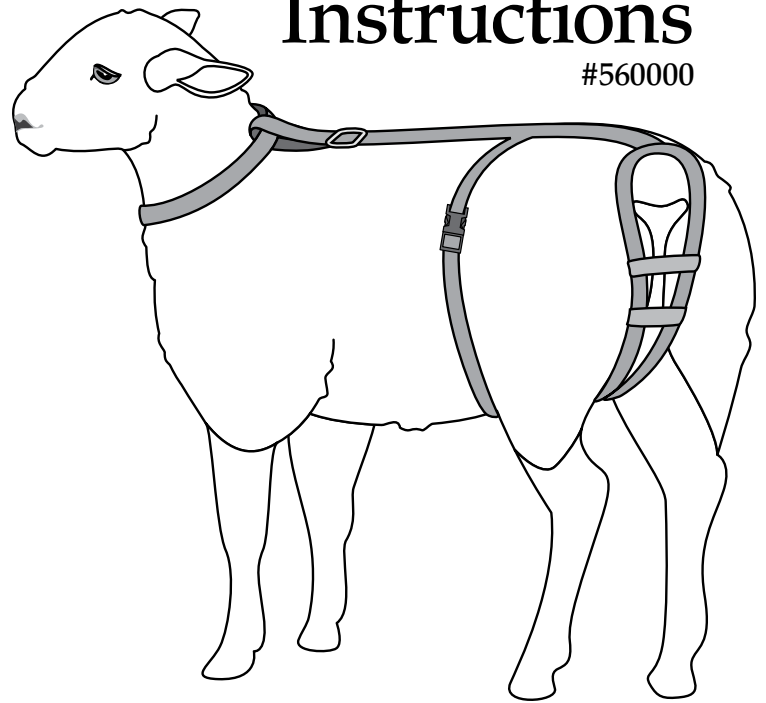

800-282-6631

[www.premier1supplies.com](http://www.premier1supplies.com)


Washington, IA

© Copyright 2016, Premier1Supplies. All rights reserved.

- Snap connections for quicker and easier fitting.
- Fully adjustable.
- Machine washable.

## Fitting Instructions:

1. Lay Harness along back of ewe, buckle towards head.
2. Attach neck strap at base of neck and tighten so comfortable.
3. Place tail through top hole.
4. Vaginal opening just appears through lower hole.
5. Take the two rear leg straps & pass between ewe's back legs, one on each side of the udder and snap into buckles on back strap; ensure that you do not trap the udder. Tighten the leg straps so they grip firmly.

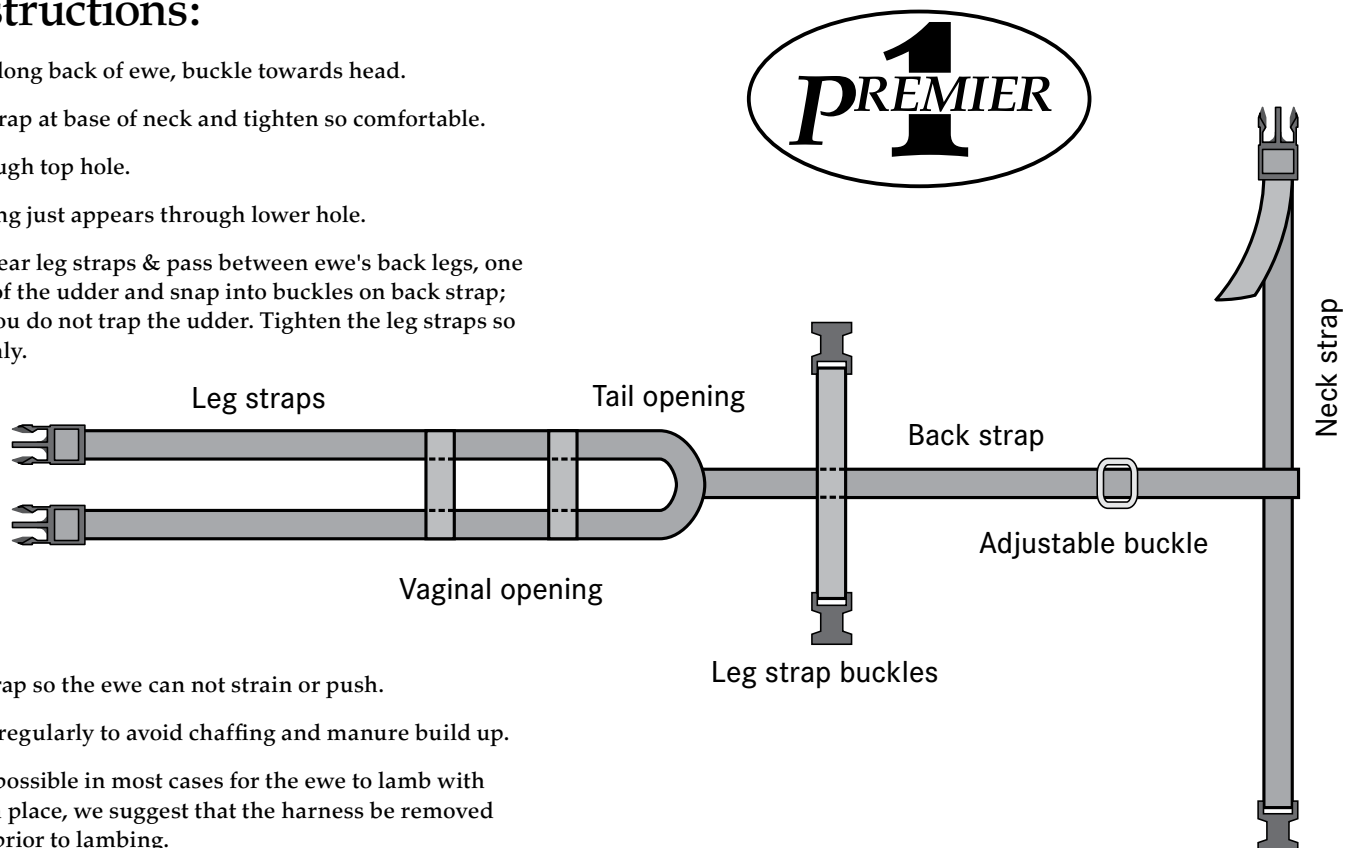

6. Adjust back strap so the ewe can not strain or push.
7. Check tension regularly to avoid chaffing and manure build up.
8. Although it is possible in most cases for the ewe to lamb with the harness in place, we suggest that the harness be removed immediately prior to lambing.