

Premier's Deck Chair Instructions


Premier's deck chair holds sheep reliably in a relaxed position. Allows the use of both hands to trim feet, tag ears, examine and medicate. Restrains heavily pregnant ewes without injury.

Users can walk away for more medicine or tools. Adjusts for width to hold all sizes of sheep (even just shorn ones) and goats.

Galvanized steel pipe and nylon webbing are strong enough to hold 500 lbs. Can be folded flat for easier storage and transporting. Shipped folded to save costs.

Assembling instructions

1. First cut the black bands. Then loosen the four nuts on the bolts on the four clamps.
2. Slide each "cross pipe" through two clamps, thus forming an adjustable rectangle with a cross pipe at both the bottom and top. However, as you do so, interweave the white mesh onto the cross pipe, so that the cross pipe will support the mesh when the animal is inside it. To show you exactly where the pipe must go, we have attached a string to each clamp and interwoven it through the mesh. Simply push the pipe down the path indicated by the string.
3. Do this at both top and bottom of the chair, and then use a 7/16 wrench to tighten the four bolts onto the clamps so that the frame is rigid.

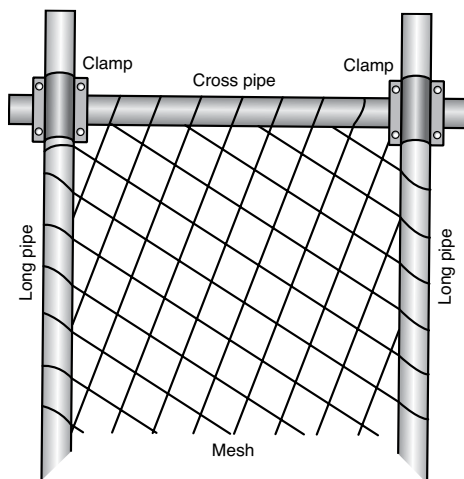


Adjusting the frame width

To adjust the width simply loosen the nuts on the clamps of one side or both.

The two long sides of the steel chair frame can then slide in and out to suit the width requirements of your animals whether they are large, small, long-wooled, short-wooled, hair coated, pregnant or open. Then retighten the nuts when you have the correct width.

Note: Most first-time users are surprised how little width is actually needed for the best restraining results. It

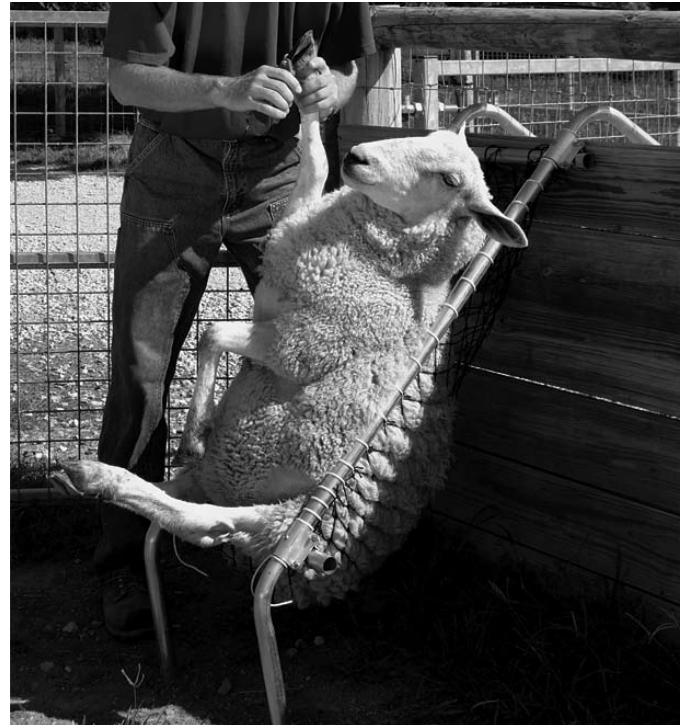

The tighter the adjustable tubular sidebars clamp the animal, the more secure the hold. If a typically-sized animal in your flock is too loose in the chair, loosen the clamps and slide the side bars closer together.


is better to keep it narrow than too wide. We suggest that you simply adjust the width to suit your needs.

At Premier we never tighten the bolt again as the chair frames rarely slide in and out too freely. Some users even screw the bolt out and throw it away.

Adjust webbing for smaller animals

We have provided enough webbing for the largest animal. For owners of large sheep (and for most adult sheep with two or more mos. of fleece) the hair webbing will probably work very well as it is. However, owners of goats and small sheep will probably prefer to reduce the depth of the "bag" created by the webbing when the chair frame is used in its narrower positions (squeezed together). To do that, untie the white nylon tie ropes on the sides, adjust the webbing to reduce the effective "bag" to the amount desired and retie it.

We have tripled the amount of "webbing" used in this animal chair to improve its ability to automatically restrain goats, small sheep and sheep with minimal wool. It is now attached to the sides as well as to each end. This means smaller, "woolless" and/or wriggly animals can no longer twist themselves out of the sides of the chair.

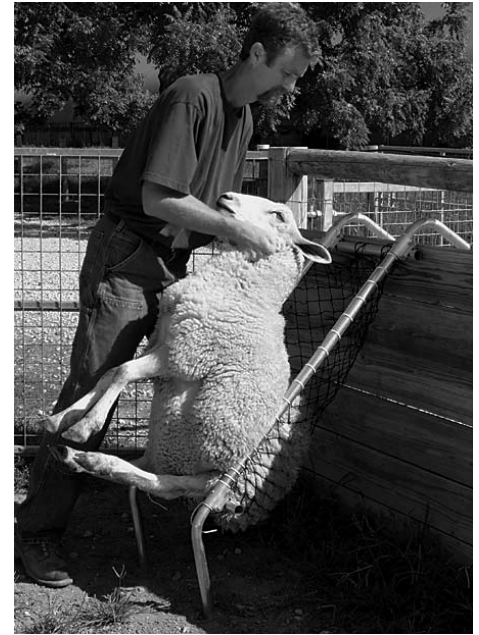

You neither need, nor should, lift sheep into a chair. Instead push it backwards by the neck/head until the butt end falls into the chair. Then lift up the head and tip it back into position. Using a small pen instead of a large one will make catching the sheep much easier. Also helps to have the chair near the corner of the pen.

To load (see photos above):

1. Best location is in the corner of a small pen—as this allows catching the animal with the least effort. Drop the top of the chair over a gate or fence at 45° angle.

2. Catch the sheep by the head. Back the sheep into the chair. As the back legs hit the bottom crossbar, the rump (and most of its weight) will fall into the webbing. (Take care not to catch the rear legs in the 2 vertical chair legs.)

NOTE: Recently added to the deck chair design are two small pegs near the top. What are these for? As an animal backs into the lower crossbar, the pegs “stop” the deck chair from sliding up the gate/panel it is leaning against. Thus you are now able to easily tip your sheep into the proper position.

3. Complete the process by lifting the head upwards into the sitting position.

How wide should the chair be?

This varies with every flock or herd, but it is usually narrower than new users expect. We suggest leaving about 14 – 15 inches between the long pipes at the beginning, and then adjusting thereafter to the frame size of your animals. Once adjusted, you will probably rarely need to change it.

What if part of the chair is accidentally damaged beyond repair? Call us. Premier will supply any part of this chair as a replacement. And since we don't like those who “rip us off” with outrageous prices for parts, we promise not to do so to you.

Which side should I leave the excess cross pipe? The excess can be on either right or left side. It's best to leave the excess on the side from which you do not normally work, so it may be opposite for left vs. right handed people.



Need replacement webbing?

If your webbing breaks or is accidentally cut, replacement webbing is available from Premier (part #807600). For lesser damage, many people use a strong piece of string to strengthen the area.

To unload:

Tip the chair, with the sheep inside it, forward with one hand. Allow the animal to fall out onto its feet. Hang onto the chair or you may be chasing the animal/chair combination around the pen.



800-282-6631
www.premier1supplies.com

Washington, IA