




# Applicator Instructions for Keto and Cal-Gel

www.premier1supplies.com • 800-282-6631 • Washington, IA

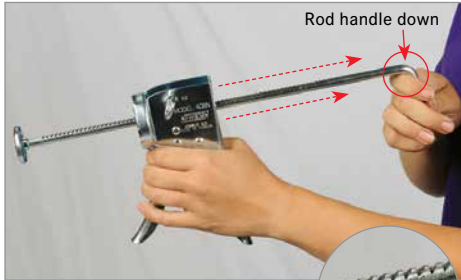


1. Turn incremented rod so the notches face upward. Pull rod away from handle.



2. Twist and lock tube into place.



3. Turn incremented rod so notches are downward.

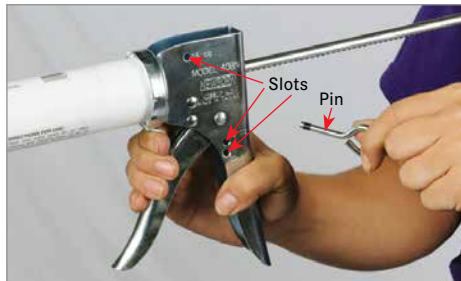

4. Pull pin and insert into desired dosage slot (5, 10 or 15cc).



5. Remove cap from gel tube.



6. Pull trigger to dispense dose.



*Applicator and gel assembled.  
Both sold separately.*

© Copyright 2016, Premier1Supplies.  
All rights reserved.