

NETTEX

NETTEX

PROLAPSE HARNESS

Fitting instructions

1. Create a loop at the end of the widest webbing strap (back strap) by sliding the plastic buckle up or down.
2. Pass the single loose strap through the loop and around the neck of the ewe. This loose strap clips back to itself once around the neck and holds the harness in place at the head end.
3. Run the wide webbing strap down the back towards the prolapse, passing the tail through the "A" shaped gap above the horizontal stitched webbing. These stitched horizontal pieces create a square to support the prolapse and should be situated top and bottom of the vulva.
4. The two thinner straps pass down between the legs either side of the udder and go forward and up to buckle up on top of the ewe at her loin.
5. If a spoon is needed it should be placed in the ewe and tied into the wool before putting the harness on.
6. Leg, back and neck straps are all fully adjustable by threading the webbing through the buckles before placing in the buckle sheath. All buckles are fully clipped in when a loud click is heard.

