



CLEANING INSTRUCTIONS

1.) **LINE CLEANING:** Before disassembling the vaccinator for cleaning, clean the line. Partially fill a sink or small container with hot, soapy water. Hold the remote draw-off spike underneath the water in one hand while holding the vaccinator in the other. Continuously squeeze the vaccinator, pumping water through it until clean. To rinse, repeat with clean water.

REMOTE
DRAW-OFF
SPIKE

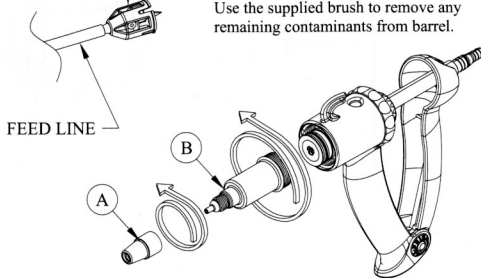

2.) **BARREL REMOVAL:** Unscrew the needle nut (A) from the barrel (B). Next, unscrew the barrel from vaccinator. Wash the all parts in hot, soapy water, then rinse. Use the supplied brush to remove any remaining contaminants from barrel.

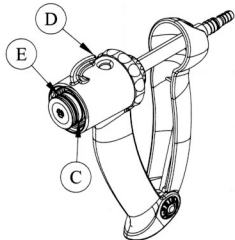

3.) **HANDLE CLEANING:** Remove the feedline from the vaccinator. At this point, the remaining handle assembly(D) should be washed or allowed to soak in hot soapy water, then rinsed and allowed to dry. Before reinstalling the barrel assembly, apply 1 or 2 drops of food-grade lubricant to the piston o-ring (E) and felt wiper ring (C).


4.) **REASSEMBLY:** Reinstall the needle (A) nut onto the barrel (B) and resinstall barrel onto the handle assembly (D). **HAND-TIGHTEN ONLY! DO NOT USE TOOLS TO TIGHTEN BARREL AS THIS MAY CAUSE BARREL BREAKAGE.**

!!CAUTION!! DO NOT OVER TIGHTEN NEEDLE NUT! FINGER TIGHTEN ONLY! OVERTIGHTENING CAN CAUSE BARREL BREAKAGE! DO NOT PLACE ANY PART OF THE VACCINATOR IN A MICROWAVE OVEN OR NEAR A HEAT SOURCE!

*PRIMA TECH
CLEANING KIT
PART# 340849.*

To learn more about these and other quality Prima Tech, USA product lines, please visit our website at www.primatechusa.com or call 910-296-6116.

PRIMA MARC
0-2ml & 0-6ml
Capacities

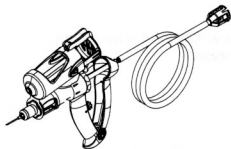

PRIMA MARC-BMV
0-2ml & 0-6ml
Capacities




PRIMA REPEATER
0-2ml, Charge Capacity
.1ml, .2ml mechanical increments




BMV
0-2ml & 0-6ml
Capacities




VACCINATOR
0-2ml & 0-6ml
Capacities

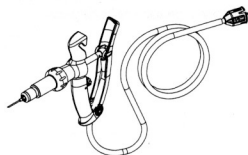

POUR-ON
0-50ml & 0-70ml Capacities

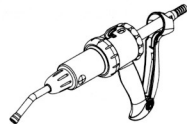

Prima Tech instruction form #340245-A

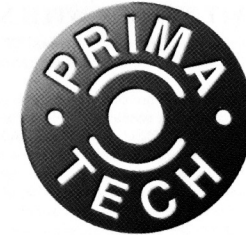

Prima Tech USA
P.O. Box 336
277 Faison McGowan Rd.
Kenansville, NC 28349
910-296-6116
www.primatechusa.com

INSTRUCTION MANUAL

INJECTOR

