

# How to Use AMBUGREEN™ Gel Barrier



Ambugreen is a physical gel barrier designed for topical use in livestock. Its unique formulation creates a hydrophobic, flexible barrier that protects the site of application from flies and environmental contamination. It is antibiotic-free, non-colonizable, and supports natural healing.

Follow these best practices to get the most from Ambugreen:

## PREPARE THE SITE

Gently wipe away any dirt, debris, or excessive fluid if present, and pat the area dry with clean disposable material.

Before applying Ambugreen, ensure that the skin or tissue is dry. Moisture can prevent the gel from adhering properly.



## APPLY THE PRODUCT

Open the tube and squeeze a small amount of Ambugreen onto a clean, gloved finger. Spread the gel over the affected area ensuring a thin, even layer that covers the site as well as a good margin of healthy tissue.





## ALLOW IT TO DRY

Ambugreen will be dry to touch within 60 seconds. You can accelerate the curing time by misting the area with water.

If the area will be in contact with an abrasive material such as sand, we advise giving the product a few more minutes to fully cure.



## RE-APPLICATION

It naturally flakes, and wears off after 48-72 hours, so there is no need to manually remove the product.

You can reapply Ambugreen directly over what is left of the previous application.



Scan the QR code  
to watch our full  
application guide



**NoBACZ**  
Healthcare

For more information about Ambugreen visit  
[www.nobacz.com/ambugreen](http://www.nobacz.com/ambugreen) or contact your local distributor.