Q-flex Ear Tags & Applicator Instructions

Fitting Instructions:

- 1. Place the female part of the tag completely under the spring fingers of the applicator. If numbered, they will be down and the tag will be extending away from the tagger. Slide the male tag over the metal pin. If numbered, they will be up and the tag will extend away from the tagger.
- 2. Before you try to insert the tag into the ear, gently squeeze the applicator together to ensure that both the male and female tags are aligned.
- 3. Place the ear between the jaws of the applicator, positioning the male tag on the back side of the ear. Make sure the tag is correctly place as shown.
- 4. Grip the ear quickly and firmly at the proposed position of the tag. Place the point of the tag in contact with the ear close to your thumb. This allows you to feel the vein therefore avoiding it. Completely close the applicator until the tag is inserted through the ear and the female side of the tag.



2031 300th St. • Washington, IA 52353